


## **BioTherm 7 Dry Ice Packing Instruction**

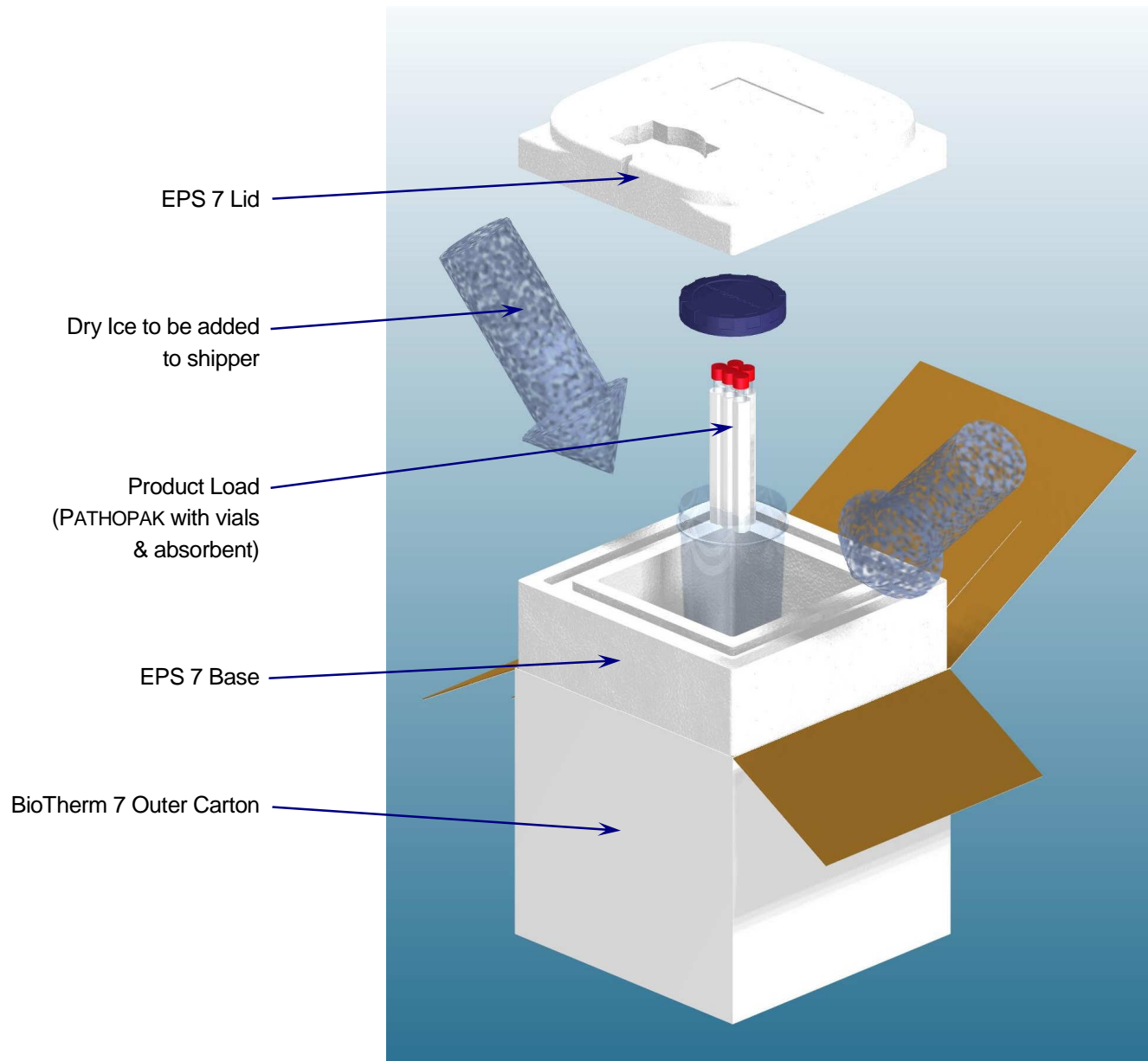

### **Caution**

Please observe all appropriate local safety precautions when handling and dispensing Dry Ice (Solid CO<sub>2</sub>)

### **Assembly Instructions**

Place the product inside the PATHOPAK and seal by screwing the lid on firmly.

Place the PATHOPAK in the centre of the EPS 7 base. Fill the EPS 7 base to the top with Dry Ice (Solid CO<sub>2</sub>) pellets.

Apply the EPS 7 lid, then close and seal the Outer Carton. The system is now ready to be shipped.