


BioTherm 10 Dry Ice Packing Instruction

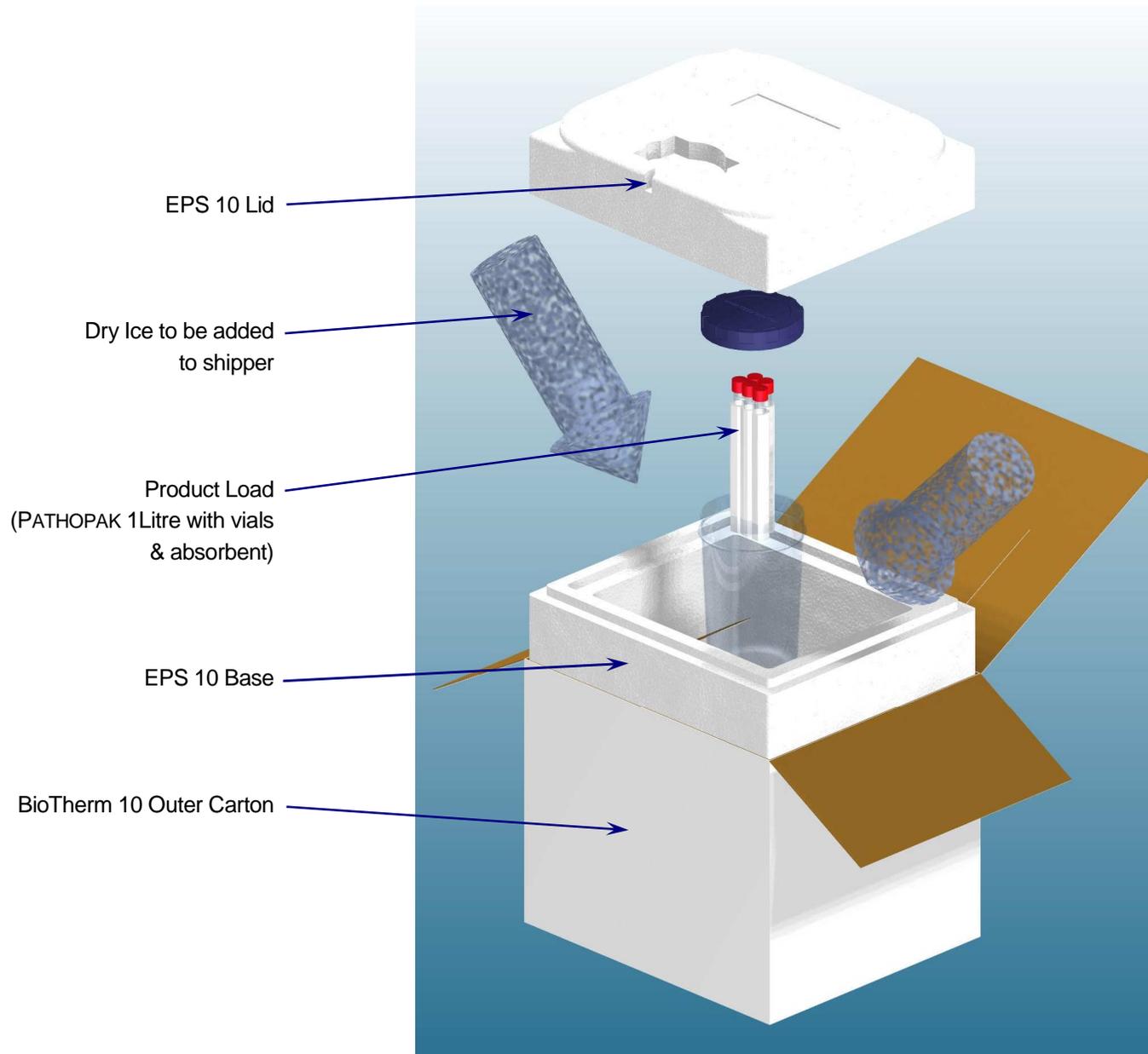

Caution

Please observe all appropriate local safety precautions when handling and dispensing Dry Ice (Solid CO₂)

Assembly Instructions

Place the product inside the PATHOPAK and seal by screwing the lid on firmly.

Place the PATHOPAK in the centre of the EPS 10 base. Fill the EPS 10 base to the top with Dry Ice (Solid CO₂) pellets.

Apply the EPS 10 lid, then close and seal the Outer Carton. The system is now ready to be shipped.