


## **BioTherm 10 Dry Ice Packing Instruction**

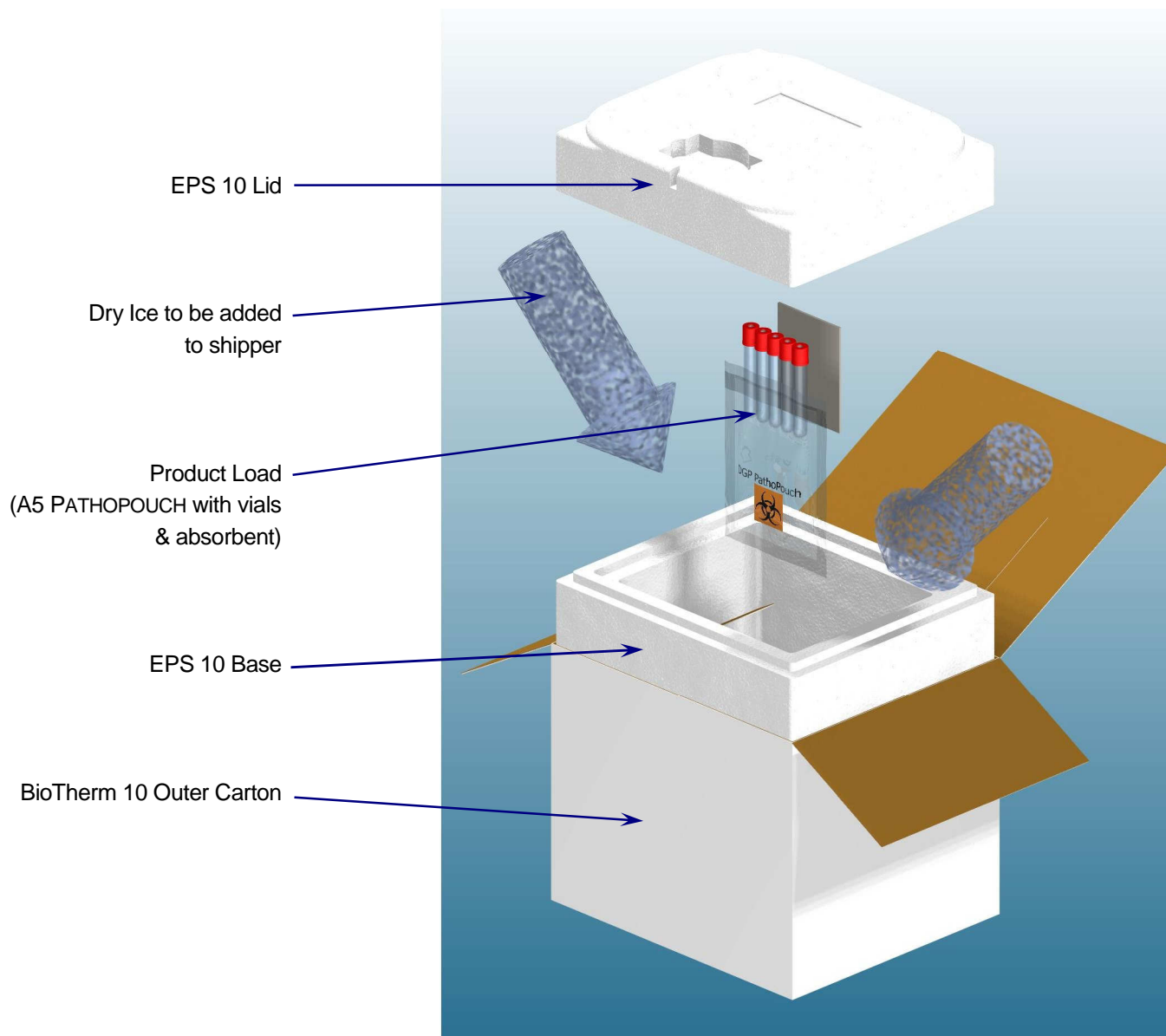

### **Caution**

Please observe all appropriate local safety precautions when handling and dispensing Dry Ice (Solid CO<sub>2</sub>)

### **Assembly Instructions**

Place the product inside the PATHOPOUCH and seal by removing the foil strip and pressing the opening of the pouch together firmly.

Place the PATHOPOUCH in the centre of the EPS 10 base. Fill the EPS 10 base to the top with Dry Ice (Solid CO<sub>2</sub>) pellets.

Apply the EPS 10 lid, then close and seal the Outer Carton. The system is now ready to be shipped.