


## **BioTherm 80 Dry Ice Packing Instruction**

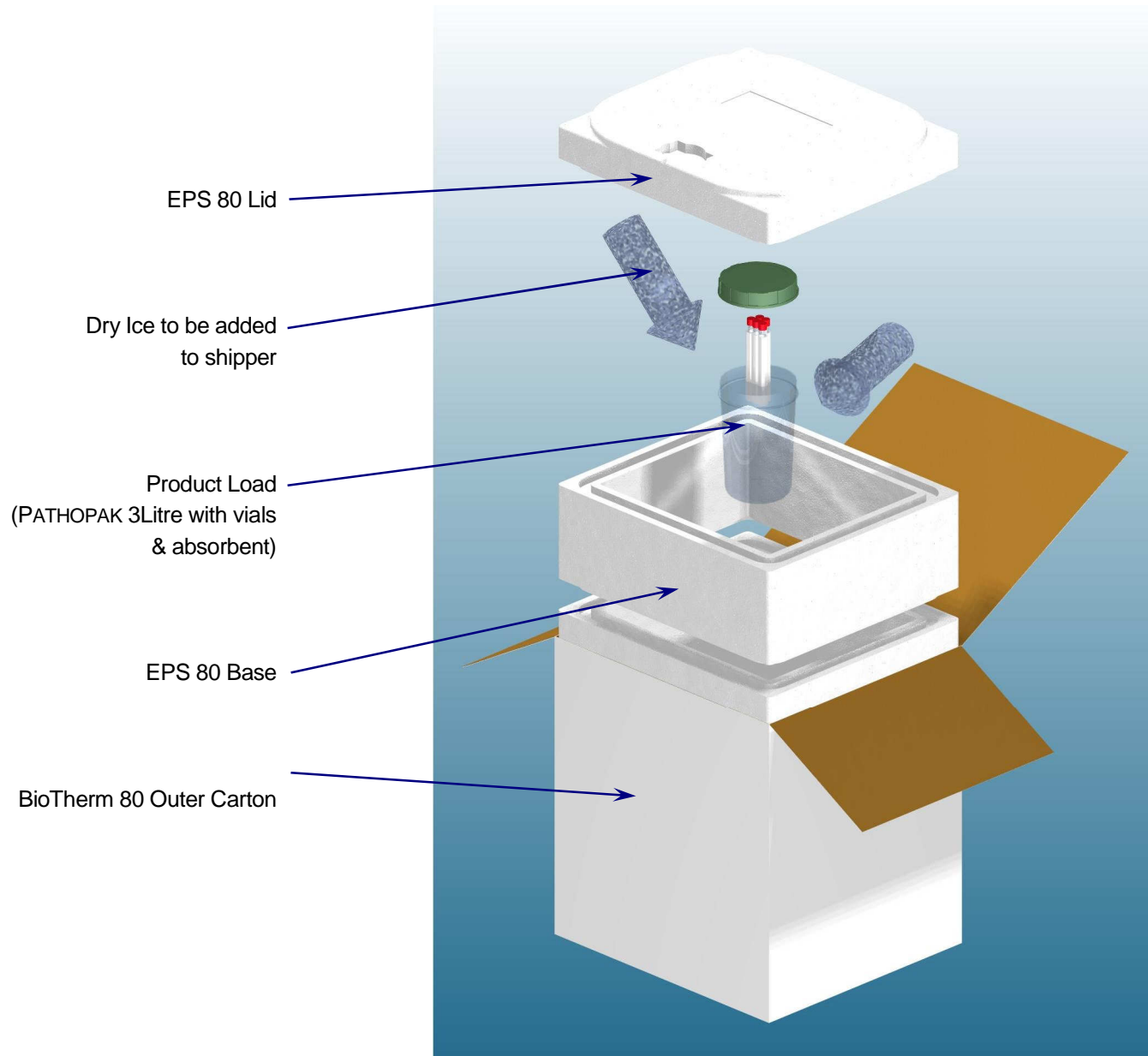

### **Caution**

Please observe all appropriate local safety precautions when handling and dispensing Dry Ice (Solid CO<sub>2</sub>)

### **Assembly Instructions**

Place the product inside the PATHOPAK and seal by screwing the lid on firmly.

Place the PATHOPAK in the centre of the EPS 80 base. Fill the EPS 80 base to the top with Dry Ice (Solid CO<sub>2</sub>) pellets.

Apply the EPS 80 lid, then close and seal the Outer Carton. The system is now ready to be shipped.